

V-Sampler®

Hematology sampler



Recommended for Vision Hema® blood cell image analysis system as well as for routine microscopy



- Standardized process for high-quality blood smear preparation
- Simultaneous preparation of 2 smears
- Setting of thickness and size of the smear
- Compact and lightweight
- Simple and easy to use

Quality blood smear in only 3 simple steps



1

Place a drop of blood on the target area



2

Depress the lever carefully but firmly and release it



3

Get a clean, thin film containing a large area

Specifications

Method of operation	manual
Length of the smear	adjustable
Number of slides proceeded simultaneously	2
Periodical adjustment	not required
Periodical lubrication	not required
Operation principle	mechanical
Power	not required
Operation temperature	15–35 °C
Operation humidity	25–80 %
Storage temperature	–50 – +55 °C
Storage humidity	25–93%
Size	150x90x140 mm (LxWxH)
Weight	0.4 kg

Ordering information

V-Sampler® Hematology sampler	80.0001.00
---	------------

Please check the intended use and the registration in your local regulation. Depending on the regulations of the respective region, some products may be used for research purpose only.

Pre-classification has to be validated by the specialist reviewing the galleries of cells. We reserve the right to change specification without notice.



West Medica Produktions- und Handels-GmbH
Brown-Boveri-Straße 6, B17-1
2351 Wiener Neudorf, Austria
tel.: +43 (0) 2236 892465, fax: +43 (0) 2236 892464
vienna@westmedica.com, www.wm-vision.com

Official Distributor